
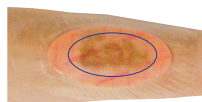
# PolyMem®: The Pink Dressing®

## DIRECTIONS FOR USE:

### APPLICATION

- 1. INITIAL DRESSING APPLICATION:** Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
- Select a dressing so that the membrane is approximately 1/4" – 2" (0.6 – 5 cm) larger than the wound. **HELPFUL HINT:** For dry wounds, moisten dressing slightly with sterile water or saline prior to application.
- Apply the dressing film side out and/or printed side out.
- 4. DRAMATIC** increase in wound fluid may be observed during the first few days. This is not uncommon, and indicates that the dressing is working. **HELPFUL HINT:** Outlining the wound size on the outside of the dressing helps in determining when a dressing change is needed.
- Keep the dressing **DRY** and in place when bathing. Change the dressing if it gets wet.
- Topical treatments are not recommended in conjunction with PolyMem family of dressings.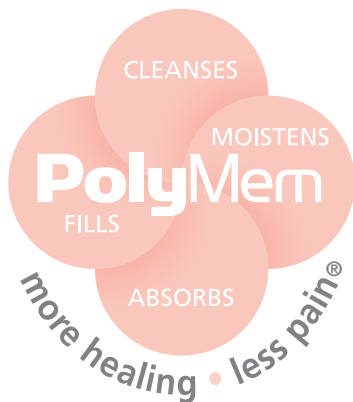

### DRESSING CHANGE

- Change dressings before the wound fluid, visible through the dressing, reaches the wound margin. Change the dressing immediately if the wound fluid reaches the edge of the membrane pad. For mildly exudating wounds in otherwise healthy patients, dressing can be left on for a recommended maximum of seven days. As with other dressings, more frequent changes may be indicated if the patient has a compromised immune system, diabetes, or infection at the wound site.
- Gently peel the adhesive film toward the center of the dressing.
- Remove the dressing from the wound site. The PolyMem formulated pad will not adhere to the wound bed, usually assuring pain-free dressing changes.
- Do not disturb the wound bed. Do not clean the wound bed or flush with saline or water unless the wound is infected or contaminated. PolyMem formulation dressings contain a wound cleanser and leave no residue. Additional cleaning of the wound may injure regenerating tissue and delay wound healing. The intact skin around the wound may be cleansed while leaving the wound bed undisturbed.
- Apply a new dressing.

Instructions for Use found in each product box should be referred to for specific product information and use.

Ferris Mfg. Corp. 5133 Northeast Parkway, Fort Worth, TX 76106 USA | 800.POLYMEM (765-9636) | [www.polymem.com](http://www.polymem.com)

Unless otherwise indicated, all trademarks are owned by or licensed to Ferris.  
© 2012 Ferris Mfg. Corp. MKL-009, Rev-11, 0912

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