

TIME TO TAKE PAIN OUT OF THE EQUATION


NURSICARE[®]
freeing moms to be moms



PAIN SHOULD NEVER COME BETWEEN YOU AND YOUR NEWBORN.

Designed by PolyMem® medical dressing experts, Nursicare® therapeutic breast pads are clinically proven to quickly and effectively alleviate pain and promote the healing of sore, irritated or cracked nipples. Containing baby-safe healing ingredients, the pads are activated by the mother's milk, triggering the natural healing process.

RELIEVES PAIN

Nursicare helps reduce the sensitivity of overactive nerve cells, effectively lessening the number of "pain messages" that are relayed to the brain and delivering soothing pain relief.

QUICK HEALING

Nursicare pads support natural healing and provide optimal conditions for restoring skin to health. The pad ensures that the healing process is not interrupted, protects the nipple from friction with clothing, keeps the area clean, and helps prevent infection.

MOM & BABY SAFE


Contains baby-safe healing ingredients. Nursicare pads have met appropriate international standards for safety and effectiveness. Have passed regulatory review and clearance in all the countries where marketed.

Pain-free breastfeeding means comfortable moms.




EASY TO USE

1

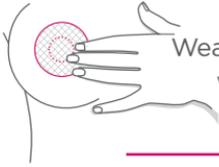


Wear the Nursicare pads inside of the bra, with printed side facing the bra.

2

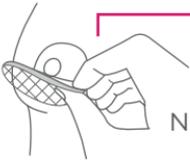

While breastfeeding your baby, take the Nursicare pad out and place it on a clean surface.

3

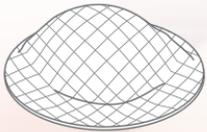

You can use the same pad several times until the liquid in the pad becomes visible through the printed side of the pad and before the absorbed milk reaches the edge of the pad.

* Please read the instructions for use inside the box before using the pad.

For more information visit our website
www.nursicare.com or www.polymem.com.



NURSICARE[®]
freeing moms to be moms

Pain should never come between a mom and her newborn. That's why we designed Nursicare therapeutic breast pads for breastfeeding moms. It quickly soothes and alleviates pain associated with breastfeeding, while rapidly healing the cracked or sore nipple.

To order Nursicare® therapeutic breast pads go to:
order.polymem.com or call **817-900-1301**


NURSICARE®
freeing moms to be moms



MKL-714-US R1 1217

© 2017 Ferris Mfg. Corp.

Ferris Mfg. Corp. 5133 Northeast Parkway, Fort Worth, TX 76106-1822 • Toll Free USA: 800-765-9636 • International: +1 817-900-1301 • www.polymem.com