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Dressing Change:

1. For an exuding wound, fluid will become visible through the top. Change before fluid reaches the wound margin, or when good practice dictates, or after no more than 7 days. Change immediately if fluid reaches the edge of the membrane pad.
2. More frequent changes may be needed due to a compromised immune system, diabetes, infection, hypergranulation, maceration, tissue, or when desiring to facilitate the removal of non-viable tissue.
3. Gently remove the dressing. The PolyMem pad will generally not adhere to the wound bed, usually assuring pain-free dressing changes. Inspect the dressing and wound to assure removal of all material.
4. If coagulated blood or dried exudate has caused the dressing to adhere to skin or hair, use a small amount of saline or water to slowly gently soften and loosen. Clean the intact skin around the wound.
5. Manage dressing changes according to protocol or as directed by a physician or other ordering clinician. In most cases, when using PolyMem, there is no need to disturb or cleanse a wound during changes unless the wound is infected or contaminated. PolyMem is designed to continuously clean the wound and does not leave residue that needs to be removed.

Excessive cleaning may injure regenerating tissue and delay wound healing.

6. Apply a new dressing.

Observations during use or change:

- During the normal healing process, the wound may appear to become larger during the first few dressing changes because the body is naturally removing non-viable tissue. If the wound continues to become larger, consult an expert.
- PolyMem dressings help support new blood vessel formation so it is not uncommon to see blood-stained wound fluid and dressings during changes.
- Be alert for signs and symptoms, such as, increased redness, inflammation, pain, warmth, odor, whitening, fragility, excessive tissue formation, or lack of healing. Consult a professional for problem situations, such as infection, irritation, sensitivity, allergy, maceration, or hypergranulation.

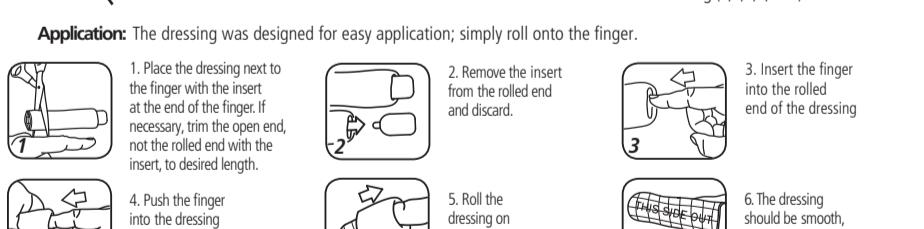
Any serious incident that has occurred in relation to this device should be reported, by the user and/or patient, to Ferris Mfg. Corp. and the competent authority of the Member State in which the user and/or patient is established.

PolyMem® Silver Finger/Toe Dressing

The dressing provides a soothing wrap shaped to fit.

Application: The dressing was designed for easy application; simply roll onto the finger.

The grid and words should be on the outside of the dressing.

The dressing should be snug, but maintain comfort. If it feels too tight, remove and apply a larger size. Remove by rolling off, opposite of application. The dressing may be applied in different ways. For toes the dressing may be cut along the sides creating flaps that can be laid upon the top and bottom of the foot. These can be secured with tape, or the dressing may be cut to form a ring or sleeve over the injured portion of the finger.

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