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PolyMem® Silver Pad or Roll Dressing

The proven multifunctional dressing.

EN

DESCRIPTION:

PolyMem polymeric membrane dressings are made of moisture-loving polyurethane matrix that contains (1) safe wound cleanser, (2) glycerin moisturizer, and (3) superabsorbents. Dressings have a thin polyurethane backing which breathes and helps keep out contamination. Dressings may be used as a primary, secondary, or combined primary and secondary dressing. PolyMem MAX™ dressings have the same polymeric membrane as PolyMem dressings, but MAX is thicker and has higher MVTR to handle more fluid.

PolyMem Silver products incorporate very fine silver particles, and will not stain skin. Silver acts inside the dressing to protect from microbial contamination. Approximate silver content by area (µg/cm²) PolyMem Silver 124 PolyMem Silver MAX™ 207.

Drug-free PolyMem dressings help to:

- effectively manage and heal wounds
- absorb fluid and provide a moist healing environment
- minimize and relieve persistent and procedural wound pain
- reduce edema, bruising, and the spread of inflammation into surrounding undamaged tissues

INDICATIONS:

Under the direction of a healthcare professional, for the management of serious wounds, such as most partial and full thickness wounds, including:

- Pressure ulcers (stages I-IV)
- Venous stasis ulcers
- Acute wounds
- Leg ulcers
- Donor and graft sites
- Skin tears
- Diabetic ulcers
- Dermatologic disorders
- 1st and 2nd degree burns
- Surgical wounds
- Vascular ulcers

PRECAUTIONS:

- For external use only
- PolyMem dressings are not compatible with oxidizing agents (hydrogen peroxide or hypochlorite solutions)
- Prior to performing imaging procedures, determine if the dressing should be removed
- Avoid contact with electrodes or conductive gels
- Topical treatments are not recommended in conjunction with PolyMem dressings
- Be alert for signs of infection or maceration
- Do not use and discontinue use on people who show signs of sensitivity, irritation, or allergy from the dressing or its materials
- Intended for one use on one person. Do not reuse. Reuse may degrade properties, spread contamination, and increase risk of infection.

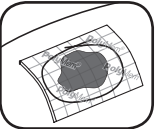
PolyMem dressings may be used when signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.

PolyMem Silver dressings are recommended when a wound displays delayed healing, signs of infection, or increased bioburden; or the patient has infection risk factors (such as contaminated wound, poor nutrition, diabetes, history of infection, etc.)

Store at room temperature ≤30°C (86°F). Excursions permitted ≤55°C (131°F) but should be minimized. **Dispose** as typical for biohazardous waste. **Dimensions** are approximate.

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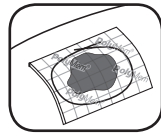
DIRECTIONS:

Initial Application:

- Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
- Select a dressing with the membrane pad ¼" – 2" (0.6 – 5 cm) larger than the wound. The dressing should also cover any inflamed or damaged area surrounding the wound. Dressings may be cut.
- For dry wounds, moisten dressing slightly prior to application.
- Apply film side out (so printing is visible).
- Secure dressings, such as with tape, netting, or gauze.
- Outline the wound on top of dressing to help determine when to change.

Use:

- A dramatic increase in wound fluid may be observed during the first few days due to the fluid attraction. This is not uncommon and indicates that the dressing is working.
- Keep the dressing dry and in place when bathing. Change the dressing if it gets wet.

Dressing Change:

- For an exuding wound, fluid will become visible through the top. Change before fluid reaches the wound margin, or when good practice dictates, or after no more than 7 days. Change immediately if fluid reaches the edge of the membrane pad.
- More frequent changes may be needed due to a compromised immune system, diabetes, infection, hypergranulation, macerated tissue, or when desiring to facilitate the removal of non-viable tissue.
- Gently remove the dressing. The PolyMem pad will generally not adhere to the wound bed, usually ensuring pain-free dressing changes. Inspect the dressing and wound to assure removal of all material.

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© 2017, 2019 Ferris Mfg. Corp.® or ™ indicate trademark