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
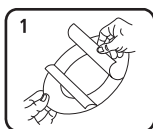
FMC-5847,R1 0426
AW-0633,R01 0426

(EN)

PolyMem® Dressing with Silicone Adhesive Border

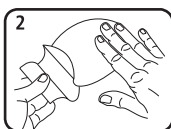Step 1. Remove one side of adhesive release liner.

Step 2. Apply over wound. Slowly remove remaining release liner(s) while pressing adhesive to the skin.

DESCRIPTION

PolyMem polymeric membrane dressings are made of moisture-loving polyurethane matrix that contains (1) safe wound cleanser, (2) glycerin moisturizer, and (3) superabsorbents. Dressings have a thin polyurethane backing which breathes and helps keep out contamination. Dressings may be used as a primary, secondary, or combined primary and secondary dressing. PolyMem Silicone Adhesive Dressings have an adhesive border that is gentle to the skin and repositionable.

Drug-free PolyMem dressings help to:

- effectively manage and heal wounds
- absorb fluid and provide a moist healing environment
- minimize and relieve persistent and procedural wound pain
- reduce edema, bruising, and the spread of inflammation into surrounding undamaged tissues

INDICATIONS FOR ADULT AND PEDIATRIC POPULATIONS:

Over-the-counter, for management of minor wounds, such as:

- Abrasions • Lacerations • Cuts • Burns

Under the direction of a healthcare professional, for

the management of serious wounds, such as most partial and full thickness wounds, including:

- Open and closed pressure injuries (ulcers) (stages 1-4, unstageable, deep tissue) • Venous stasis ulcers • Acute wounds
- Leg ulcers • Donor and graft sites • Skin tears • Diabetic ulcers
- Dermatologic disorders • 1st and 2nd degree burns • Surgical wounds • Vascular ulcers • May be used as part of a comprehensive prevention and treatment bundle

PRECAUTIONS

- For external use only • PolyMem dressings are not compatible with oxidizing agents (hydrogen peroxide or hypochlorite solutions) • Prior to performing imaging procedures, determine if the dressing should be removed • Avoid contact with electrodes or conductive gels • Topical treatments are not recommended in conjunction with PolyMem dressings • Be alert for signs of infection or maceration • Do not use and discontinue use on people who show signs of sensitivity, irritation, or allergy from the dressing or its materials • Intended for one use on one person. Do not reuse. Reuse may degrade properties, spread contamination, and increase risk of infection • For adult and pediatric populations. Not recommended for neonate use.

The use of dressings as part of a prophylactic therapy does not preclude the need to continue to develop and follow a comprehensive pressure injury prevention protocol.

PolyMem dressings may be used when signs of infection are present if proper medical treatment that addresses the cause of the infection has been implemented.

Store at room temperature ≤30°C (86°F).

Excursions permitted ≤55°C (131°F) but should be minimized.

Dispose as typical for biohazardous waste.

Dimensions are approximate.

DIRECTIONS

Initial Application

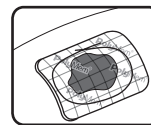
1. Prepare the wound according to protocol or as directed by a physician or other ordering clinician.
2. Select a dressing with the membrane pad ¼" – 2" (0.6 – 5 cm) larger than the wound. The dressing should also cover any inflamed or damaged area surrounding the wound.

3. For dry wounds, moisten dressing slightly or leave wound moist after preparation. For good adhesion, ensure skin is clean and dry.

4. After application, outline the wound on top of dressing to help determine when to change.

5. Minimize repositioning. However, if needed, gently remove and reapply. Do not contaminate adhesive border with oil or cleaning agents.

6. If used for prevention, ensure the membrane pad covers the area at risk of pressure injury. Be aware that damage can extend beyond the area of direct deformation or contact.

Use

1. A dramatic increase in wound fluid may be observed during the first few days due to the fluid attraction. This is not uncommon and indicates that the dressing is working.
2. Keep the dressing dry and in place when bathing. Change the dressing if it gets wet.
3. When used for prevention: Minimize lifting edges for skin assessment. Leave in place for up to 7 days and change as needed.

Dressing Change

1. For an exuding wound, fluid will become visible through the top. Change before fluid reaches the wound margin, or when good practice dictates, or after no more than 7 days. Change immediately if fluid reaches the edge of the membrane pad.
2. More frequent changes may be needed due to a compromised immune system, diabetes, infection, hypergranulation, macerated tissue, or when facilitating the removal of non-viable tissue.
3. Dressing Change: Lift the edge of the dressing and remove. Apply new dressing.

4. If coagulated blood or dried exudate has caused the dressing to adhere to skin or hair, use a small amount of saline or water to slowly gently soften and loosen. Clean the intact skin around the wound.

5. Manage dressing changes according to protocol or as directed by a physician or other ordering clinician. In most cases, when using PolyMem, there is no need to disturb or cleanse a wound during changes unless the wound is infected or contaminated. PolyMem is designed to continuously clean the wound and does not leave residue that needs to be removed.

Excessive cleaning may injure regenerating tissue and delay wound healing.

Observations during use or change

- During normal healing, the wound may appear larger during the first few dressing changes because the body is naturally removing non-viable tissue. If the wound continues to become larger, consult an expert.
- PolyMem dressings help support new blood vessel formation so it is not uncommon to see blood-stained wound fluid and dressings during changes.
- Be alert for signs and symptoms, such as: increased redness, inflammation, pain, warmth, odor, whitening, fragility, excessive tissue formation, or lack of healing. Consult a professional for problem situations, such as: infection, irritation, sensitivity, allergy, maceration, or hypergranulation.

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© 2026 Ferris Mfg. Corp.® or ™ indicate trademark