
<p>Do not use if pouch has been damaged or opened. Nicht verwenden, wenn der Beutel beschädigt oder geöffnet ist. Använd inte skadad eller öppnad förpackning. Ikke bruk hvis forpakningen er skadet eller åpenet. Non utilizzare se la busta è stata aperta o danneggiata. Ne pas utiliser si l'emballage du pansement a été ouvert ou est endommagé. No utilizar si el estuche está dañado o abierto. Eğer poş zarar görmüş veya açılmış ise kullanmayın.</p>		<p>Keep away from sunlight and heat. Vor Sonnenlicht und Hitze schützen. Håll borta från solljus och värme. Ikke utsett produktet for sollys og varme. Tenere lontano dalla luce solare e dal calore. Garder à l'abri des rayons du soleil et de la chaleur. Mantener fuera del alcance de la luz solar y el calor. Güneş ışığı ve ısıdan koruyun.</p>	
<p>Store ≤30°C (86°F). Bei ≤30°C (86°F) lagern. Förvara i ≤30°C (86°F). Lagre ved ≤30°C (86°F). Conservare a ≤30°C (86°F). Conserver à une température ≤30°C (86°F). Conservar a ≤30°C (86°F). ≤30°C (86°F) ve altındaki sıcaklıklarda saklayın.</p>		<p>Not made with natural rubber latex. Nicht aus Naturkautschuklatex hergestellt. Produkten är inte tillverkad av naturgummillatex. Ikke laget av naturlig lateks. Privo di lattice di gomma naturale. Ne contient pas de latex naturel. No están hechos de látex de caucho natural. Doğal kauçuk lateksten üretilmemiştir.</p>	
		<p>Single use. Do not re-use. Zum Einmalgebrauch. Nicht wiederverwenden. För engångsbruk. Återanvänd inte. Til engangsbruk. Ikke bruk om igjen. Monouso. Non riutilizzare. À usage unique. Ne pas réutiliser. Desechables. No reutilizar. Tek kullanımlıdır. Tekrar kullanmayın.</p>	

by **PolyMem®**
SPORTSWRAP®

FERRIS MFG. CORP.
5133 Northeast Parkway
Fort Worth, TX 76106-1822 U.S.A.
Toll Free U.S.A.: 800-765-9636
International: +1 817-900-1301
www.polymem.com
© 2026 Ferris Mfg. Corp.® or ™ indicate trademark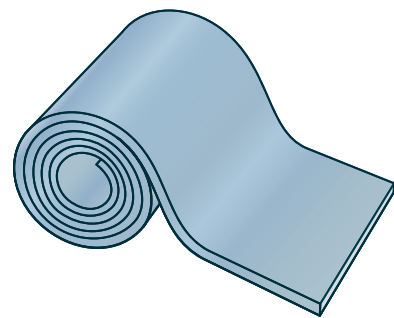

Emergo Asia Pacific Pty Ltd
201 Sussex Street
Darling Park, Tower II, Level 20
Sydney, NSW 2000 Australia

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

AW-0540_R02 0326
FMC-5994_R7 0326

EN

ROLL Long-Lasting, Drug-Free Pain Relief

PRODUCT DESCRIPTION

SportsWrap is a patented, hydrophilic, polyurethane membrane pad. The semi-permeable continuous thin film backing is optimized for moisture vapor permeability.

INDICATIONS

For the local management of swelling, inflammation, bruising, stiffness or pain due to acute or chronic injury when the skin is intact.

PRECAUTIONS

Consult your physician for all serious injuries, and if symptoms persist after wearing or increase during use. Not sterile. Not for broken skin or open wounds. For external use only.

PREPARATION

Clean the skin of dirt, topical medications and lotions. Residual moisture on skin is beneficial.

APPLICATION

Apply as soon as possible after injury. SportsWrap provides flexibility of application.

No matter which method of application you use or which part of the body you choose to wrap, the following steps will apply:

- Step 1** Unroll the green pad with "THIS SIDE OUT" away from the skin.
- Step 2** Wrap in a continuous motion overlapping slightly.
- Step 3** Secure pad with athletic or adhesive tape.

Wrap firmly for close contact with the skin but apply gently to maintain comfort. The green pad must maintain intimate contact with the skin to work effectively.

TRIMMING

Depending on anatomy and wrapping technique, the SportsWrap may be too long. The green pad can be trimmed with a pair of scissors if necessary. Return the unused portion to the resealable pouch. Overlap to ensure complete coverage.

WEARING

- Wear all day and night for 5 days.
- Avoid wetting the wrap. Remove before bathing or showering.
- If the green pad becomes saturated with sweat, discard and replace. The wrap cannot be "dried out" and reused.
- Another SportsWrap may be worn for an additional 5 days. Continued use will help promote continued comfort.
- Keep wearing even when you feel better!
- Adjust periodically to maintain contact and comfort.

STORAGE

Keep dry and store at room temperature and humidity.

Any serious incident that has occurred in relation to this device should be reported, by the user and/or patient, to Ferris Mfg. Corp. and the competent authority of the Member State in which the user and/or patient is established.

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